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[illegible]

Tip merača glukoze	Kompatibilnost sa Extraneal (ikodekstrin) PD rastvorom (glukoza- specifičan)	Tip testa*	Proizvođač
Cobas Pulse	Da	GDH-FAD	
TRUE METRIX	Da	GDH-FAD	Trividia Health www.trividiahealth.com Telefon: N/A
TRUE METRIX GO	Da	GDH-FAD	
TRUE METRIX Air	Da	GDH-FAD	
TRUEresult ³	Ne preporučuje se	GDH-FAD	
TRUEresult Twist 2 ³	Ne preporučuje se	GDH-FAD	
TRUEyou ³	Ne preporučuje se	GDH-FAD	
Rightest Blood Glucose Monitoring System GM700	Da	GDH-FAD	Bionime Corporation www.bionime.com Telefon: N/A
Rightest Blood Glucose Monitoring System GM700 Pro	Da	GDH-FAD	
Rightest Blood Glucose Monitoring System GM700S	Da	GDH-FAD	
Rightest Blood Glucose Monitoring System GM700SB	Da	GDH-FAD	
Rightest Blood Glucose Monitoring System GM700 Pro 2	Da	GDH-FAD	
Rightest Blood Glucose Monitoring System GM720	Da	GDH-FAD	
Rightest Blood Glucose Monitoring System GM722	Da	GDH-FAD	
Rightest Blood Glucose Monitoring System Max	Da	GDH-FAD	
Rightest Blood Glucose Monitoring System Max Plus	Da	GDH-FAD	
Rightest Blood Glucose Monitoring System Wiz	Da	GO	
Rightest Blood Glucose Monitoring System Wiz Plus	Da	GO	
Rightest Blood Glucose Monitoring System GM550	Da	GO	
Rightest Blood Glucose Monitoring System GM280	Da	GO	
Rightest Blood Glucose Monitoring System GM280B	Da	GO	
Rightest Blood Glucose Monitoring System GT333	Da	GO	
Rightest GT333 Blood Glucose Monitoring System	Da	GO	
Rightest Blood Glucose Monitoring System GM300	Da	GO	
Rightest Blood Glucose Monitoring System GM250	Da	GO	
Rightest Blood Glucose Monitoring System GM260	Da	GO	
GE100 Blood Glucose Monitoring System	Da	GO	

Tip merača glukoze	Kompatibilnost sa Extraneal (ikodekstrin) PD rastvorom (glukoza- specifičan)	Tip testa*	Proizvođač
Diathrive+ Blood Glucose Monitoring System	Da	GO	
Diathrive+ Bluetooth Blood Glucose Monitoring System	Da	GO	
iGlucose Blood Glucose Monitoring System	Da	GO	
GE333 Blood Glucose Monitoring System	Da	GO	
Quintet AC Blood Glucose Monitoring System	Da	GO	
Mylife Pura Blood Glucose Monitoring System	Da	GO	
Mylife Unio Blood Glucose Monitoring System	Da	GDH-FAD	
Mylife Unio Neva Blood Glucose Monitoring System	Da	GDH-FAD	
Mylife Unio Cara Blood Glucose Monitoring System	Da	GDH-FAD	
LineaD SIRIO Blood Glucose Monitoring System	Da	GO	
LineaD EVO Blood Glucose Monitoring System	Da	GO	
LineaD ORO HR Blood Glucose Monitoring System	Da	GDH-FAD	
LineaD ORO Blood Glucose Monitoring System	Da	GDH-FAD	
LineaD ORO24 Blood Glucose Monitoring System	Da	GDH-FAD	
Rightest Blood Glucose Monitoring System Max Tel	Da	GDH-FAD	
Rightest Blood Glucose and Ketone Monitoring System Max Dual	Da	GDH-FAD	

¹ Ovi Abbott merači (zajedno sa Dexcom G4 Platinum, G5 Mobile, G6 i Medtronic iPro2) su vodeći sistemi za kontinuirano praćenje glukoze (CGM) i od strane proizvođača se **ne preporučuje** njihova primena kod pacijenata na dijalizi i pored toga što se princip rada zasniva na GO metodi, trenutno nema definitivnih naučnih dokaza o njihovoj kompatibilnosti sa lekom Extraneal. Konsultovati proizvođača za dodatne informacije.

² ACCU-CHEK Nano (ne Aviva ili Performa) i ACCU-CHEK Aviva Plus sistemi su dostupni jedino u SAD, i koriste test trake koje su brendirane kao ACCU-CHEK SmartView i ACCU-CHEK Aviva Plus. Ovi sistemi koriste maltoza nezavisne Mut Q-GDH (**kompatibilne**) trake. Konsultovati proizvođača za dodatne informacije.

³ TRUEresult (GDH-FAD), TRUEresult Twist 2, TRUEyou merači nisu testirani sa Baxter-ovim preporučenim graničnim vrednostima za maltozu i ikodekstrin. Prema tome ovi merači **nisu preporučeni** za upotrebu kod pacijenata na dijalizi.

Tip testa* (Test Type)

GO = glukoza-oksidaža [glucose oxidase]

GDH-NAD = glukoza-dehidrogenaza sa nikotinamid-adenin dinukleotidom [glucose dehydrogenase with nicotinamide-adenine dinucleotide]

GDH-FAD = glukoza-dehidrogenaza sa flavin-adenin dinukleotidom [glucose dehydrogenase with flavin-adenine dinucleotide]

Mut Q-GDH = glukoza-dehidrogenaza sa piroloninolinonim, modifikovan da eliminiše interferenciju sa maltozom [glucose dehydrogenase with pyrroloquinolinequinone modified to eliminate maltose interference]

Molimo vas da pogledate kompletne informacije o propisivanju leka EXTRANEAL (ikodekstrin), rastvor za peritonealnu dijalizu.

Baxter i EXTRANEAL su robne marke kompanije Baxter International Inc.

Druge robne marke ili proizvodi koji se ovde pojavljuju, vlasništvo su odgovarajućih kompanija.